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harex

USER MANUAL / MANUAL DE USO /
MANUEL D'UTILISATION / GEBRAUCHSANWEISUNG

Device for
the control of male
urinary incontinence

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EN **ATTENTION**
Before using the Device, read the
instructions carefully.

INTENDED USE

Harex® is a medical device for the control male urinary incontinence. It is a mechanical device for external use. It's made of medical grade material, latex-free.

DESCRIPTION

Harex® has been developed with the aim of improving the quality of life of men suffering from urinary incontinence. Its design takes into account the anatomy of the penis, so that by placing Harex® behind the glans (head of the penis) and by gentle pressure on the urethra, it prevents and controls involuntary urine leakage.

The proper placement of Harex® is with the Urethral Pressure Pad (D) on the lower part of the penis shaft, as this is where the urethra is located. At the same time, Harex® allows blood circulation thanks to the flexible design of the belt, which encircles the penis at the top, and due to the lack of pressure on the sides of the penis.

Harex® is made of medical grade plastic materials and tested not to absorb liquids.

It does not contain metallic elements, which allows passing through metal detection systems (i.e. those used in airports or train stations).

The set includes a plastic case (washable) for transporting and/or storing the device.

SIZE

- Outer dimensions: 48 mm x 54 mm (4,8 cm x 5,4 cm)
- Belt Width: 18 mm (1,8 cm)
- Distance between "D" (urethral pressure pad) in its upper position & the Belt (C): 8 mm (0,8 cm)
- Weight: 18 g

INSTRUCTIONS FOR USE

WHERE TO PLACE:

- Place it along the shaft of the penis. **Recommendation:** place it just behind the head of the penis (glans). Fig1
- **Important:** The correct placement of the urethral pressure pad (D) is underside of the penis, where the urethra is located.

A. HOW TO PUT THE DEVICE

1. Move down the Belt Adjustment Lever (B) and open the Belt (C).
2. Put the penis into the device.
3. Adjust the Belt (C) to the diameter of the penis.
4. Move upward the Belt Adjustmet Lever (B) to lock the Belt (C).
5. To block the urethra, move the urethral pressure activator (A) to its upper position.

B. TO URINATE

- 6.1 Unblock the urethra by moving the urethral pressure activator (A) to its lower position.
- 6.2 Urinate.
7. Upon completion, to block the urethra move the urethral pressure activator (A) to its upper position.

C. HOW TO REMOVE THE DEVICE

8. Move the urethral pressure activator (A) to its lower position.
- 9.1 Move down the Belt Adjustment Lever (B) and open the Belt (C).
- 9.2 Loosen completely the Belt (C) to remove the device.

Optional: The belt (C) has a numerical ruler to mark its optimal position and thus facilitate its adjustment.

WARNINGS AND PRECAUTIONS

- Urinate every 2-3 hours and/or based on the physiological needs of each user.

- It is recommended to rest the penis during sleeping hours by removing Harex®. In case overnight use of Harex® is necessary, loosen the strap to a lighter pressure.
- Incorrect and/or excessive use of Harex® may cause skin irritation and/or discomfort and/or slight pain.
- In case of discomfort and/or slight pain, make sure that the device is correctly placed. Also, ensure there are no creases in the skin of the penis. In case of continued discomfort, try a looser fit.
- Do not use Harex® if you have open wounds and/or cutaneous irritation on the skin of the penis.
- In case of repeated urethral strictures, consult your doctor before using Harex®.
- For effective use, the length of the penis shaft should be >2 cm.

CLEANING

Wash the device by hand with soap and warm water.

TERMS AND CONDITIONS

You are using Harex® at your own risk. If you have a medical problem other than incontinence, consult your doctor before using Harex®. If you experience problems, stop using it immediately and inform your doctor.

It is recommended to replace Harex® after 12 months of use.

For more information, we invite you to visit our website at: www.harex.net.

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